suavinēx care

**Manual de instrucciones
Instruction manual
Manuel d'instructions
Manuale d'istruzioni
Gebrauchsanweisungen
Manual de instruções**

**APP
CONTROL**

ES	ADVERTENCIAS	4 - 7
EN	WARNINGS	7 - 11
FR	AVERTISSEMENTS	11 - 15
IT	AVVERTENZE	15 - 18
DE	HINWEISE	19 - 23
PT	ADVERTÊNCIAS	23 - 27

ES	MANUAL DE INSTRUCCIONES	30 - 38
EN	INSTRUCTION MANUAL	39 - 47
FR	MANUEL D'INSTRUCTIONS	48 - 56
IT	MANUALE D'ISTRUZIONI	57 - 65
DE	GEBRAUCHSANWEISUNGEN	66 - 74
PT	MANUAL DE INSTRUÇÕES	75 - 83

Este manual está disponible en su última versión en la web www.suavinex.com / The latest version of this manual is available in our website www.suavinex.com / Ce manuel est disponible dans sa dernière version sur le site www.suavinex.com / Il presente manuale aggiornato è disponibile sul sito web www.suavinex.com / Dieses Handbuch steht Ihnen in seiner neuesten Version auf der Webseite www.suavinex.com zur Verfügung / A versão mais recente deste manual está disponível na página web www.suavinex.com

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EN: WARNINGS

Please read these instructions carefully and keep them for future reference.

1. This appliance is designed as a "heartbeat listener" for domestic use; any other use is considered

unsuitable and therefore, dangerous. The manufacturer is not responsible for any damage caused by improper, erroneous or irrational use of the appliance.

2. This product is not a medical device and, therefore, it should not be used for any medical purpose (monitoring, diagnosis, etc.). Always follow the instructions of healthcare professionals.

3. This product is not a toy. Keep out of the reach of children. Do not allow children to play with this appliance. This appliance **SHOULD NOT** be used by children, people with limited physical, sensory or mental capabilities or by people with some type of learning difficulty, unless they are supervised or have received instructions relating to the correct handling of the product and are aware of the possible dangers. All use, cleaning and maintenance must be carried out by an adult.

4. This appliance is a precision instrument that must be handled with care.

5. After removing the heartbeat

listener from its packaging, make sure it is intact and there is no visible damage. In case of doubt, do not use the appliance and contact the manufacturer or distributor. If the charging cable is damaged, it must be replaced by the manufacturer or an authorised repair service.

6. The use of any electrical appliance requires compliance with some basic rules:

- Do not pull the USB cable to disconnect the plug from the socket.
- Do not leave the appliance exposed to atmospheric agents such as rain, sun, etc.
- Keep the appliance and USB cable away from sources of heat.
- Do not touch the appliance with wet or damp hands.
- Do not immerse the appliance in water or other liquids.
- Do not touch the appliance if it has fallen into water; if this happens, do not disconnect the USB cable from the plug or thermometer. Do not use the appliance again until you have contacted the manufacturer, distributor or authorised repair centre.

7. In order to avoid potential explosions:

- Use away from environments where flammable gases and/or anaesthetics are present.

- Do not throw the heartbeat listener onto a fire.

8. Try not to cause vibrations when using the heartbeat listener.

9. Do not sterilise the appliance by any method.

10. If you have any doubt after using the device, consult your doctor.

11. Under no circumstances can this heartbeat listener replace specialised medical foetal monitoring.

12. This appliance contains no repairable parts or parts that may be reused by the consumer. Do not use accessories, parts or components that are not supplied by the manufacturer, distributor or authorised service.

13. In the event that the appliance is defective or does not work properly: switch it off immediately. Do not attempt to fix the appliance and contact the manufacturer, distributor or authorised repair centre. Any

repairs must be performed by authorised technical personnel.

14. The product's characteristics may be subject to change without prior notice.

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EN: INSTRUCTION MANUAL

CONTENTS

1. PARTS OF THE HEARTBEAT LISTENER	39
2. INSTRUCTIONS FOR USE	40
2.1. USE OF THE HEARTBEAT LISTENER	40-41
2.2. HEARTBEAT DETECTION	41-42
2.3. CHARGING THE BATTERY	42-43
3. CLEANING AND MAINTENANCE	43-44
4. TROUBLESHOOTING	44-45
5. TECHNICAL SPECIFICATIONS	46-47

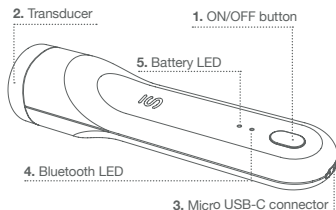
SMART HEARTBEAT LISTENER

It is now possible to listen to your baby's heartbeat from home thanks to the new Suavinex's Smart heartbeat listener: its ultrasonic technology will allow you to hear your baby's heartbeat from the sixteenth week of pregnancy. In addition, with the Suavinex Link® App, you can monitor the rate of your baby's heartbeat and record the sound so that you can share it with your family and friends.

Discover all the application's possibilities at www.suavinex.com

1. PARTS OF THE HEARTBEAT LISTENER

2. INSTRUCTIONS FOR USE

2.1. USE OF THE HEARTBEAT LISTENER

In order to use the Suavinex Smart heartbeat listener, follow these steps:

1- Switch on the heartbeat listener by pressing the button (1) for a couple of seconds. At that point, the Bluetooth light (4) will start to flash blue. This means that the device is ready to connect to the Suavinex Link® App.

• NOTE:

- If the heartbeat listener's battery is not charged, when it is switched on the battery light (5) will flash red 3 times, after which the device will switch off.

2- Enable the Bluetooth connection on your smartphone.

3- Open the Suavinex Link® App and go to the "devices" tab. All the Suavinex Smart products will appear under this tab. Select the heartbeat listener.

• IMPORTANT:

- The first time you use the heartbeat listener, you must pair it by pressing this icon . Once paired, follow the instructions in the application's wizard to complete the connection process.

- The next times you connect the heartbeat listener, as you already paired it the first time, this icon  will appear. It can be displayed in two colours: orange, meaning that the heartbeat listener is switched on and the connection is therefore activated, or grey, meaning that the heartbeat listener is switched off and the connection is not activated.

4- Once the device is connected to your smartphone, the blue Bluetooth light (4) will remain on.

5- Before starting, make sure you find a comfortable position for the transducer (2) to detect the foetal heartbeat. In section 2.2 HEARTBEAT DETECTION below we show you the heartbeat detection areas; these will differ depending what stage of pregnancy you are at.

6- Apply a generous amount of conductive gel or cream to the transducer (2) or the abdomen.

7- At the time of detection, you must bear in mind the best location, positioning the transducer according to the instructions shown in section 2.2.

8- When you start to hear the heartbeat, you will see a graph in the app with your baby's heart rate (the heart rate obtained by this heartbeat listener is approximate, so it may not coincide with the real rate).

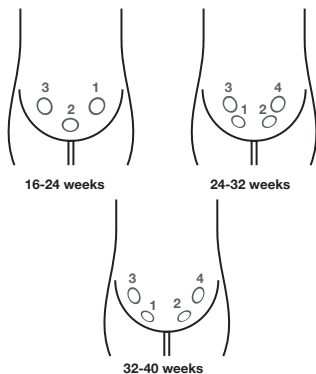
9- If you wish, you can record your baby's heartbeat by pressing the "RECORD" button. The recordings will be saved in the Suavinex Link® App and you can share them with your family and friends.

• NOTES:

- You can increase or decrease the volume of the sounds detected by using the controls on your smartphone.
- You can listen to the sound through your smartphone's speaker or by connecting a headset.
- Do not be alarmed if you do not find the heartbeat, sometimes the position of the foetus does not allow it to be heard. Try again at another time.
- If the appliance is not connected to the app, it will automatically switch off after 90 seconds of inactivity.
- When you finish using the heartbeat listener, shut it down properly in one of the following ways to avoid using up the battery:
 1. Make sure the app is completely closed (otherwise, it will carry on working in the background).
 2. You can switch off the heartbeat listener directly by pressing the button (1) for a few seconds.

2.2. HEARTBEAT DETECTION

The points where you should look for the heartbeat depend on your stage of pregnancy, as shown in the following images:

As the foetus grows, the detection areas will become further apart. This is because its position, and therefore its heart, move upwards and outwards.

For the best possible experience with the heartbeat listener, three important aspects must be taken into account:

1. A generous amount of conductive gel should be applied to the transducer (2) or to the abdomen.
2. The transducer (2) should be positioned on the abdomen and must be completely in contact with the skin.
3. You must exert a little pressure and change the orientation of the heartbeat listener at each point on the abdomen (as indicated above).

• **IMPORTANT:**

- If you do not find the heartbeat at one point on the abdomen, try the next one.
- The foetal position may change during the exploration, and therefore you will have to adjust the orientation of the heartbeat listener until you re-detect the heart activity with the most clarity possible.
- If you are in the first weeks of the first stage of pregnancy (16-20 weeks), it may be a little more difficult to detect the heartbeat.

• **NOTE:**

- The ultrasonic emission of this device is considerably lower than industrial standards; therefore, detection and sound quality are not the same as those achieved with a professional appliance.

• **RECOMMENDATIONS:**

- The best times to listen to your baby's heartbeat are:
 - 30 minutes after getting up in the morning.
 - 30 minutes before you go to bed at night.
 - An hour after eating.
 - Especially in the early stages of pregnancy, a full bladder can help improve the sounds that are heard.

2.3. CHARGING THE BATTERY

This heartbeat listener has a rechargeable battery. If it is drained, the battery light (5) will flash three times in red when the device is switched on and the unit will automatically switch off. To charge the battery, follow these steps:

1. Remove the silicone plug from the USB-C connector (3) of the appliance.
2. Connect the end of the USB-C charger cable to the connector (3).
3. During battery charging, a white battery light (5) will begin to flash. Once the battery is charged, this white light will remain constant.

• **IMPORTANT:**

- It is not possible to use the device while the battery is charging.
- Use an appropriate power supply to charge the battery of the heartbeat listener, taking into account the technical specifications shown in section 5. TECHNICAL SPECIFICATIONS.
- Do not attempt to remove the battery from the appliance.

3. CLEANING AND MAINTENANCE

Next, we will give you a series of tips that you should follow for the correct cleaning and maintenance of your heartbeat listener.

• **IMPORTANT:**

- Switch off the appliance before cleaning and/or maintenance.

1. Clean the contact area (transducer (2)) with a 70% alcohol solution or isopropyl alcohol. You may also use a cloth slightly dampened with water or soapy water. Let it air dry or use a soft cloth to dry it.

2. Before each use, clean the contact area to remove any remaining conductive gel. By doing this, the heartbeat listener will achieve greater sound detection on each use.
3. If you are going to use it after a long time, wipe the transducer area with a clean, dry cloth.
4. Do not use abrasive products such as aggressive detergents, solvents or similar products, as they can cause permanent damage to the surfaces of the appliance.
5. Do not leave the surface of the appliance in contact with any liquid substance.

4. TROUBLESHOOTING

PROBLEM	SOLUTION
THE HEARTBEAT LISTENER DOES NOT SWITCH ON	- Press the button (1) for a few seconds until the Bluetooth light begins to flash. - It is possible that the appliance needs charging, if this is the case, charge the appliance (Section 2.3 CHARGING THE BATTERY).

THE HEARTBEAT LISTENER DOES NOT CONNECT TO THE SMARTPHONE	- Check that the heartbeat listener is not switched off. If this is the case, press the button (1) for a few seconds until the Bluetooth light begins to flash. - It is possible that the appliance needs charging, if this is the case, charge the appliance (Section 2.3 CHARGING THE BATTERY). - Check that your smartphone's Bluetooth connection is activated. - Check that the distance between the device and your smartphone is less than 5 metres.
THE HEARTBEAT LISTENER DISCONNECTS FROM THE SMARTPHONE	- Check that the heartbeat listener is not switched off. If this is the case, press the button (1) for a few seconds until the Bluetooth light begins to flash. - It is possible that the appliance needs charging, if this is the case, charge the appliance (Section 2.3 CHARGING THE BATTERY). - Check that the distance between the device and your smartphone is less than 5 metres. - Check that your smartphone's Bluetooth connection is activated.
NO SOUND IS COMING FROM THE SMARTPHONE	- Check that the volume of your smartphone is turned up to a suitable level. - Check whether the headphones are connected to your smartphone. - Check that the device is connected to your smartphone. - Check that the heartbeat listener is switched on. If it is off, press the button (1) for a few seconds until the Bluetooth light begins to flash. - Check that the measurements were performed properly according to paragraph 2.2.
THE HEARTBEAT LISTENER BATTERY DOES NOT CHARGE	- Check that the cable is connected correctly to the USB-C (D) connector of the heartbeat listener. - Check that the charging cable is connected to a power source. - Check that the power source is working properly.

If you still have questions, our customer service will be happy to help you. Email: suavinex@suavinex.com

5. TECHNICAL SPECIFICATIONS

This heartbeat listener shows the FHR (foetal heart rate) value

Sensitivity: higher than 90 dB
Measurement range: from 50 to 210 beats per minute
Accuracy: ± 2 beats per minute
Operating conditions:
Temperature: from 5°C to 40°C
Humidity: below 80%
Atmospheric pressure: from 86 kPa to 106 kPa
Transport and storage conditions:
Temperature: from -10°C to 55°C
Humidity: below 93%
Atmospheric pressure: from 86 kPa to 106 kPa
Operating mode: Continuous Doppler
Water resistance: IP22
Acoustic working frequency: 2.5 MHz $\pm$ 15%
Ultrasound power: Up to 20 mW
Effective emission area: 245 mm² $\pm$ 15%
P: < 1MPa
Iob: 20 mW/cm²
Ispta: < 100 mW/cm²
Battery: 3.7 V / 400 mAh
Charging time: 2 hours
Battery life: 3 hours
Standard Bluetooth: 4.2 (compatible with standard Bluetooth 3.0 and 4.0)

Exclusivas Rimar, S.L.
C/ del Rublo, parcelas R83 - R94
03114 Alicante, España/Spain

Made in China

This appliance complies with the essential requirements established in the current EU directives.

Caution

Batch code

Serial number

Please read the instructions

Input voltage

In the EU this product should not be disposed of with other household waste in order to avoid harming the environment or human health. It must therefore be recycled correctly, promoting the sustainable reuse of material resources. To

return the used appliance, please use the designated recycling systems or contact the establishment where you purchased the product. They will dispose of this product appropriately so that it can be safely recycled.

This product complies with EU Directive 2012/19/EU. The crossed-out wheeled bin symbol on the appliance indicates that the product must be disposed of separately from other household waste at the end of its useful life, either by taking it to a selective collection centre for electrical appliances or by returning it to the distributor when you purchase another similar appliance. The user is responsible for taking the appliance to a special waste disposal centre at the end of its useful life. If the unused appliance is correctly collected as separated waste, it can be recycled, processed and eliminated ecologically. This prevents it having a negative impact on the environment and health and helps ensure that the product materials are recycled. For more information on available waste disposal services, contact your waste disposal office or the shop where you bought the appliance.

IP22

Protection of electrical equipment against the entry of external actors: dust or water.

Google is a registered trademark of Google LLC.

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Google é uma marca registada de Google LLC.
LABORATORIOS SUAVINEX, S.A